GOhearin

USER GUIDE • BEDIENUNGSANLEITUNG
MODE D'EMPLOI • ISTRUZIONE D'USO

Welcome

Welcome to a world of better hearing!

We understand that hearing loss can have an adverse effect on quality of life. That is why we are committed to building hearing aids of the highest quality. Your new hearing aid will soon become indispensable to you, and make your life more fulfilled and enjoyable.

Your new GOhear-in hearing aid has been programmed to match your individual hearing loss profile to help enhance your hearing and so improve your ability to understand and follow conversations.

A period of adjustment may be necessary to become used to your new hearing aid and the restored world of sound you will experience. The adjustment time varies for each individual and may last as little as two weeks or as long as several months. Allow yourself time to become familiar with sounds you have not heard for a long time.

Before using your hearing aid, please read this user guide thoroughly.

Table of Contents

Welcome	1
Safety Information	3
GOhear-in Diagram	5
Fitting the Correct Tip	6
Left and Right Hearing Aids	7
Turning Your Hearing Aid On	7
Turning Your Hearing Aid Off	7
Inserting and Removing Your Hearing Aid	8
Selecting Your Program	8
Battery Information	10
Inserting the Battery	11
Cleaning Your Hearing Aid	12
Caring for Your Hearing Aid	14
Extended Storage	15
Telephone Use	15
Troubleshooting Guidelines	16
Warranty Information	18
Returning for Service	19
Declaration of Conformity	19
Interpretation of Serial Number	20
Key to CE symbols	20

Safety Information


Keep your hearing aid out of the reach of children. Hearing aids contain small pieces that may be swallowed and may present a choking hazard. Contact your physician immediately if any piece is swallowed.

- Read all of the information in the User Guide before using the hearing aid.
- Always follow the directions of your Sonetik Specialist in fitting and placing the device in your ear.
- If you experience any discomfort, discontinue use and contact your Sonetik Specialist or physician.
- Failure to change the tip according to the instructions provided could result in injury to your ear.
- Be careful when removing ear wax. Using cotton swabs or other small objects can damage the ear canal. Contact your physician.
- DO NOT push your hearing aid too deep into your ear. In the rare cases in which an ear tip or other part becomes lodged in your ear canal, seek medical attention for removal.
- DO NOT leave your hearing aid on a radiator, near a stove, on a sunny window ledge, in a car or in any other hot place. Excessive heat and cold can damage the hearing aid.
- DO NOT wear the hearing aid when using a hair dryer at home or at the beauty salon, spa or hairdresser's as this will damage the hearing aid.

- DO NOT wear the hearing aid during medical examinations such as MRI, X-rays or CT scans. Different types of radiation may damage the hearing aid. Consult your medical professional prior to any medical examinations.
- DO NOT apply hair spray when wearing your hearing aid. This will damage the hearing aid.
- DO NOT wear the hearing aid when in contact with water, for example, when taking a bath or shower, washing your hair, swimming or during a spa treatment. Allowing water into any opening of the hearing aid will damage the hearing aid.
- DO NOT store the hearing aid in the bathroom or near a steam room. Moisture created by the steam can damage the hearing aid.
- DO NOT immerse your hearing aid in water or other liquids or put it in contact with detergents, solvents or other liquids. This will damage the hearing aid.
- DO NOT allow water or any liquid to enter any openings of the hearing aid. This will damage the hearing aid.
- DO NOT drop your hearing aid and avoid any sudden impact. This will damage the hearing aids.
- Dispose of the batteries, device, and components in accordance with local regulations. Unlawful disposal may cause environmental pollution.

- DO NOT use your hearing aid if damaged.
- DO NOT attempt to modify the shape of the tip by yourself.
- It is recommended that you remove the battery and store your hearing aid overnight in a hearing aid dehumidifier.
- Always carry spare batteries for immediate replacement.
- Replace the battery as soon as you hear the low battery warning signal of 3 double beeps.

GOhear-in Diagram

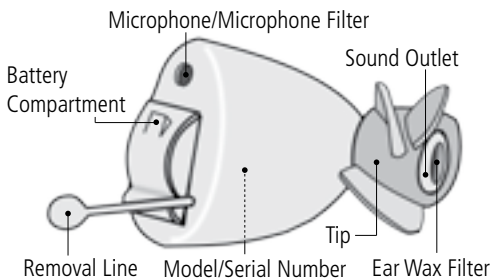Standard on GOhear-in1 / GOhear-in2 / GOhear-in3 / GOhear-in4

Fitting the Correct Tip

Each hearing aid is available with a choice of three tips.

You will notice a tip has already been fitted to your hearing aid. If, after properly inserting your hearing aid into your ear, you experience whistling when the volume is turned up, or it is too loose or tight, you, or your Sonetik Specialist, can exchange this tip for one of a different size. Before removing the tip, please take note as to how it was fitted, so that when you replace it you will know the correct placement on the nozzle.

NOTE: Once you find the size tip that best suits your ear, fill in the appropriate section on the hearing test result sheet that you were given after your hearing consultation. This will serve as a reminder when it becomes necessary to re-order replacement tips.

Left and Right Hearing Aids

If you are wearing hearing aids in both ears, it is important to use the correct device for each ear. The hearing aid for the right ear has a «red» shell and for the left ear a «blue» shell.

Turning Your Hearing Aid On

To turn your hearing aid «on», you simply need to close the battery door after inserting a battery.

Turning Your Hearing Aid Off

Apply light pressure on the notch on the battery door, and swing the door open until the battery is fully visible. Your hearing aid is now «off».

When the hearing aid is not in use, it is recommended that you always open the battery door fully. This helps prevent excessive battery drain and allows proper ventilation of the hearing aid.

Inserting and Removing Your Hearing Aid

This hearing aid should fit comfortably and easily in your ear. Your Sonetik Specialist will provide instructions on proper insertion and removal.

To insert a hearing aid, grasp the removal line between your thumb and forefinger, so that the sound outlet of the hearing aid points at you and the product information printed on the hearing aid is facing away from you. Move your hand to the ear canal entrance and gently push the hearing aid into your ear canal until the faceplate covers it up. When properly inserted it should not stick out of the ear canal entrance.

To remove your hearing aid, locate and grasp the removal line between your thumb and forefinger and slowly pull the hearing aid from your ear.

CAUTION: In the unlikely event that the tip should come off and remain in your ear, go immediately to your physician to have it removed. NEVER insert anything into your ear in an attempt to remove the tip.

Selecting Your Program

Refer to the hearing test result that you received from your Sonetik Specialist to find the base program for your

right and left GOhear-in hearing aids.

Your GOhear-in hearing aid is equipped with a “no-touch” program change system. To change the program, tap twice onto your ear, and you will receive confirmation that the program has been changed.

There are four different programs (volume levels). With each set of taps you select a higher program, unless your hearing aid is already set to program 4. From program 4, further tapping will bring it back to program 1 again.

If you change to program 1, you will hear 1 beep

If you change to program 2, you will hear 2 beeps

If you change to program 3, you will hear 3 beeps

If you change to program 4, you will hear 4 beeps

Whenever you turn your GOhear-in hearing aid on, it start at the last program selected and the applicable beep sequence will be audible after approx. 10 seconds.

You may find that certain programs are more comfortable in different situations. For instance program 1, which has less amplification, might be more suitable for noisy environments such as restaurants. On the other hand, program 4, which has more amplification, might be more useful when listening to a speaker in an auditorium.

Battery Information

Battery Size

Your GOhear-in hearing aid uses a standard size A10 hearing aid battery, which is available for purchase from your Sonetik Specialist.

Low Battery

Your GOhear-in hearing aid is equipped with a low battery detection system, which continuously monitors battery life. When the battery level becomes low, a low battery warning sequence consisting of 3 double beeps is initiated. When the warning beeps have finished, normal operation resumes. Battery replacement is recommended when you hear this low battery warning signal.

The low battery warning sequence will repeat every 10 minutes until the battery becomes critically low. The end of battery life warning sequence, consisting of 6 double beeps, is then triggered. The device is then muted and put into low power sleep mode. At this point it is necessary to replace the batteries.

The hearing aid battery packaging usually shows a voltage. This is the idle voltage of the battery (when not in use). If you measure the voltage with a voltmeter, you will see the battery voltage in use, which is much lower than when it is idle. A lower result from the voltmeter does not limit the performance or service life of the battery in any way.

Inserting the Battery

Step 1: Apply light pressure on the notch on the battery door, and swing the door open. Do not force it. Remove the battery. If you have difficulties removing the small battery with your fingers, use a pen nib to push it out of its compartment.

Step 2: To ensure proper battery usage, please remove the tag from the battery and allow the battery to sit for two minutes before inserting it into the hearing aid. Insert the battery carefully with the «+» side up, facing the side where the product information is printed.

Step 3: Swing the door into the closed position. The door should close easily. Do not force it. If there is resistance, check that the battery has been inserted correctly.

The life of your hearing aid battery will vary depending on length of daily use, listening environments, and how your hearing aid is programmed. When the hearing aid battery goes dead, remove it immediately and replace it with a new battery. Batteries may be purchased from your Sonetik Specialist.

Cleaning Your Hearing Aid

The following instructions will help you to extend the life of your hearing aid.

To Clean the Exterior of the Hearing Aid:

You should inspect your hearing aid daily and clean it periodically to control build-up of dust, dirt and wax. To clean, wipe the surface very gently with a soft, dry cloth or use only moist cleaning wipes made especially for hearing aids. Never use water or other liquids when cleaning the hearing aid.

To Clean the Ear Tip:

Inspect the tip of the ear canal portion of the hearing aid for any wax build-up. To clean, wipe the tip very gently with a soft, dry cloth or use only moist cleaning wipes made especially for hearing aids and their accessories. The tip should be cleaned regularly and exchanged at the latest after three months.

Be very careful not to push wax into the ear wax filter.

It is important that the ear tip fits properly into your ear. Your hearing aid is constructed from the highest quality materials, and the likelihood of an allergic reaction is low. If the ear tip irritates your ear in any way, contact your Sonetik Specialist.

To Clean the Microphone Filter:

Use a brush to remove any ear wax from the microphone filter. Be careful that you don't push the ear wax inside the microphone filter. Hold the hearing faceplate facing downwards ensuring loose wax falls out and away and not into the filter. If you can't remove the ear wax from the microphone filter, contact your Sonetik Specialist.

To Clean the Ear Wax Filter:

It is important to replace the filter as soon as it has become clogged, or after one week to maintain optimal performance.

You can purchase additional filters from your Sonetik Specialist. There is a tool included in the exchange filter pack. Use this tool to pick up the clogged filter and remove it from the hearing aid. Pick up a new ear wax filter and press it onto the aperture at the end of the hearing aid. When purchasing exchange filters, you will receive a separate user guide. If you have difficulties replacing the ear wax filter, contact your Sonetik Specialist.

The wearing of an in-the-ear hearing aid may accelerate the accumulation of ear wax. This can easily be removed by your physician. Regular annual visits are recommended. You should not use cotton swabs to clean your ears, due to the risk of pushing the wax deeper into the ear canal.

Caring for Your Hearing Aid

The tip may need to be changed if it becomes stiff, brittle, kinked or damaged. It should always be changed every three months, at the very least. Your Sonetik Specialist should change the ear tip. This is to prevent the tip from detaching from the hearing aid during insertion into the ear. If your Sonetik Specialist instructs you to change the tip yourself, make sure it is securely fastened to the hearing aid before inserting it into your ear.

It is important to replace the ear wax filter as soon as it has become clogged, or after one week.

Hearing aids are exposed to moisture in the form of humidity and perspiration while being worn. The daily use of a dehumidifying kit with a drying capsule is recommended. Your Sonetik Specialist may carry these accessories for you to purchase or recommend a place of purchase to you.

Extended Storage

If it is necessary to store your hearing aid for an extended amount of time, follow the steps below for proper storage.

1. Remove the battery.
2. Clean the hearing aid according to the instructions on page 12.
3. Place the hearing aid in a cool, dry place. It is recommended that you use either the original container or a hearing aid dehumidifier.

Telephone Use

Your GOhear-in hearing aid allows you to use the phone as you would normally do – by holding the phone up to your ear. Don't press the telephone receiver too much onto your ear to avoid unwanted feedback. If feedback still occurs, a program with less amplification might be better to use on the phone.

Troubleshooting Guidelines

Understanding Speech While Eating

If you experience the sound of your own chewing and swallowing as too loud during meals, just take one hearing aid out, temporarily. The same applies if you find you are also having trouble understanding speech while eating.

No Sound

- Ensure the battery is inserted correctly and the battery compartment door is fully closed.
- Replace the battery.
- Ensure there is no wax or debris in the ear wax filter. If needed, clean according to the instructions on page 13.

Not Loud Enough

- Replace the battery.
- Ensure there is no wax or debris in the ear wax filter. If needed, clean according to the instructions on page 13.
- Ensure the hearing aid is properly positioned in the ear canal.
- Excessive earwax can prevent sound from entering the ear. Contact your physician.
- Changes to your hearing status can affect how well you hear with hearing aids. Contact your Sonetik Specialist if you suspect any change in your hearing status.

Whistling/Feedback

- Ensure the hearing aid is properly positioned in the ear.
- Excessive ear wax can cause whistling. Contact your Sonetik Specialist.

Sound is Distorted or Unclear

- Replace the battery.
- Ensure there is no wax or debris in the ear wax filter. If needed, clean according to the instructions on page 13.
- Your hearing aid may have poor battery contact.
- Return the hearing aid to your Sonetik Specialist.

Intermittent Sound

- Replace the battery.
- Ensure there is no wax or debris in the ear wax filter. If needed, clean according to the instructions on page 13.
- Your hearing aid may have poor battery contact.
- Return the hearing aid to your Sonetik Specialist.

Static Noise

- Replace the battery.
- If you are near an electromagnetic field (i.e. near a computer screen or fluorescent lights), step away to see if the noise clears.
- Your hearing aid may have poor battery contact.
- Return the hearing aid to your Sonetik Specialist.

Warranty Information

Please see your Sonetik Specialist regarding your specific warranty information. Our guarantee covers any defect in material or faulty production, except microphone and ear wax filters, tips and batteries. We will, at our discretion, repair or replace the hearing aid or any defective part, at our expense, within the limits of the warranty.

This warranty will become invalid if the hearing aid has been tampered with or altered in any way, or if the serial number has been altered, effaced, removed, or if any repairs have been made without authorisation.

Normal wear and tear, faulty handling, defects caused by lack of maintenance and cleaning or any issues caused by ear wax in the hearing aid are also not covered by the warranty.

Any implied warranty shall be limited to 24 months from the date of purchase. The only entity or individual who is authorised to perform the warranty obligations is the manufacturer. All in-house warranty work performed by the manufacturer will be done within a reasonable period of time. If warranty work is performed and there appears to be a defect, malfunction, or failure to conform to the warranty, the obligations will be met if brought to our attention within one month of the date of the original warranty work.

This warranty is limited to the scope set forth herein, and may only be extended with the explicit written agreement of Sonetik AG or its affiliated companies. The receipt will be required for any warranty.

This warranty does not include a promise of specific hearing improvement as this depends upon your audiological needs, upon proper fitting and adjustments by your Sonetik Specialist and upon proper use by the owner.

Returning for Service

You should visit your Sonetik Specialist if your hearing aid is in need of service.

Before planning to return your hearing aid for repair, please refer to the troubleshooting solutions on the preceding pages. In many cases, these solutions can prevent the time and cost associated with a repair.

Declaration of Conformity

Hereby Sonetik AG declares that this GOhear-in product meets the requirements of the Medical Devices Directive 93/42/EEC. The full text of the Declaration of Conformity can be obtained from Sonetik AG.

Interpretation of Serial Number

Each hearing aid has its own unique number located on the exterior of the aid. The first two digits of the serial number represent the year in which the hearing aid was manufactured.

Key to CE symbols

On the box, label and instructions for use you may encounter the following symbols, shown here with their meaning:

	Medical device directive CE mark and number of the notified body
	Manufactured by
	Date of Manufacture
	Please read user guide
	Serial number
	This hearing aid may not be thrown away as normal household waste. Please dispose of old or unused hearing aids at waste disposal sites intended for electronic waste, or give your hearing aid to your Sonetix Specialist for disposal. Proper disposal protects the environment and health.
	The products described in this user guide adhere to the requirements for an applied part of Type BF of EN 60601-1. The surface of the hearing aid is specified as application part of Type BF.

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