GOhearon

USER GUIDE • BEDIENUNGSANLEITUNG
MODE D'EMPLOI • ISTRUZIONE D'USO

Welcome

Welcome to a world of better hearing!

We understand that hearing loss affects one's quality of life. That is why we are dedicated to building the highest quality hearing aids. Your new hearing aid will soon become indispensable and will make your life more pleasant and more enjoyable.

Your new GOhear-on hearing aid has been programmed to your individual hearing loss profile to help enhance your hearing and so improve your ability to understand and follow conversations.

A period of adjustment may be necessary to become used to your new hearing aid and the restored world of sounds you will experience. The adjustment time varies for each individual and may last as little as two weeks or as long as several months. Allow yourself time to become familiar with sounds you have not heard for a long time.

Before operating your hearing aid, please read this user guide thoroughly.

Owner Information

Complete the following information for future reference:

Left:

Model _____

Base Program _____

Receiver Unit _____

Ear Dome _____

Serial Number _____

Right:

Model _____

Base Program _____

Receiver Unit _____

Ear Dome _____

Serial Number _____

Purchase Date _____

Battery Size _____

Original Warranty Expires _____

Sonetik Specialist

Name: _____

Business Name: _____

Address: _____

Postcode: _____ City: _____

Phone: _____

Table of Contents

Safety Information	4
GOhear-on Diagram	6
Proper Placement on the Ear	7
Left and Right Hearing Aids	8
Turning the Hearing Aid On	8
Turning the Hearing Aid Off	8
Selecting Your Program	9
Battery Information	10
Inserting the Battery	11
Cleaning Your Hearing Aid	12
Caring for Your Hearing Aid	15
Extended Storage	15
Telephone Use	15
Troubleshooting Guidelines	16
Warranty Information	17
Returning for Service	18
Declaration of Conformity	18
Interpretation of Serial Number	19
Key to CE symbols	19

English

Deutsch

Français

Italiano

Safety Information


Keep the hearing aid out of the reach of children. The hearing aid contains small pieces that may be ingested and may present a choking hazard. Contact your physician immediately if any piece is ingested.

- Read all of the information in the User Guide before using the hearing aid.
- Always follow the directions of your Sonetik Specialist in fitting and placing the device on your ear.
- If you experience any discomfort, discontinue use, and contact your Sonetik Specialist.
- Failure to change the dome according to the instructions provided could result in injury to your ear.
- Be careful when removing earwax. Using cotton swabs or other small objects can damage the ear canal. Contact your physician.
- DO NOT push the dome of your hearing aid too deep into your ear. Place the ear dome lightly inside the outer rim of your ear canal as instructed by your Sonetik Specialist. In the rare cases in which an ear dome or other part becomes lodged in your ear canal, seek medical attention for removal.
- DO NOT leave your hearing aid on a radiator, near a stove, in a sunny window, in a car or in any other hot place. Excessive heat and cold can damage the hearing aid.
- DO NOT wear the hearing aid when using a hair dryer at home or at the beauty salon, spa or hairdresser's as this will damage the hearing aid.

- DO NOT wear the hearing aid during medical tests such as MRI, X-rays or CT scans. Different types of radiation may damage the hearing aid. Consult your medical professional prior to any medical test.
- DO NOT apply hair spray when wearing your hearing aid. This will damage the hearing aid.
- DO NOT wear the hearing aid when in contact with water, for example, when taking a bath or shower, washing your hair or swimming or during a spa treatment. Allowing water into any opening in the hearing aid will damage the hearing aid.
- DO NOT store the hearing aid in the bathroom or near a steam room. Moisture created by the steam can damage the hearing aid.
- DO NOT immerse your hearing aid in water or other liquids or put it in contact with detergents, solvents or other liquids. This will damage the hearing aid.
- DO NOT allow water or any liquid to enter any openings on the hearing aid. This will damage the hearing aid.
- DO NOT drop your hearing aid and avoid any sudden impact. This will damage the hearing aid.
- DO NOT twist or turn the receiver unit when connecting it to the body of the hearing aid, as this may damage the hearing aid.
- Dispose of the batteries, device, and components in accordance with local regulations. Unlawful disposal may cause environmental pollution.

- DO NOT use your hearing aid if damaged.
- DO NOT attempt to modify the shape or length of the receiver unit by yourself.
- It is recommended you remove the battery and store your hearing aid overnight in a hearing aid dehumidifier.
- Always carry spare batteries for immediate replacement.
- Replace the battery as soon as you hear the low battery warning signal of 3 double beeps.
- Always care for your hearing aid as outlined in this user guide.
- Open the battery door fully when the hearing aid is not in use. This helps prevent excessive battery drain and allows proper ventilation of the hearing aid.
- DO NOT connect any other devices to your hearing aids while it is in use.

GOhear-on Diagram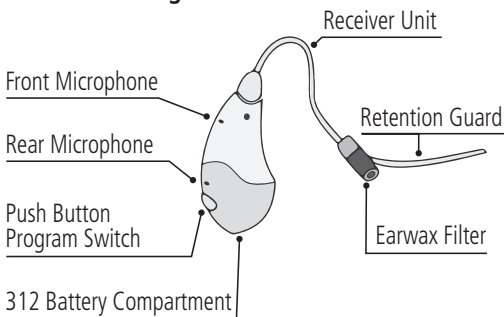

Standard on GOhear-on 1 / GOhear-on 2 / GOhear-on 3 / GOhear-on 4


Proper Placement on the Ear

Your GOhear-on hearing aid should fit comfortably and easily over the top of your ear. Your Sonetik Specialist will provide instruction on proper insertion and removal.

Step 1: Place the hearing aid over the top of your ear.

Step 2: With your index finger and thumb, hold the receiver unit where it attaches to the ear dome and gently push the dome into your ear until it is just inside your ear canal.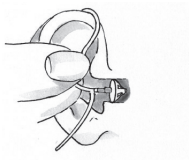

Step 3: Place the retention guard of the receiver unit into the bow of your concha.

Left and Right Hearing Aids

If you are wearing hearing aids on both ears, it is important to use the correct device in each ear. The receiver units are marked with "red" color for the right ear and "blue" color for the left ear.

Turning the Hearing Aid On

To turn your hearing aid "on", you simply need to close the battery door after inserting a battery.

Turning the Hearing Aid Off

Apply light pressure on the latch on the battery door, and open until the battery is visible. Your hearing aid is now "off".

When the hearing aid is not in use, it is recommended that you always open the battery door fully. This helps prevent excessive battery drain and allows proper ventilation of the hearing aid.

Selecting Your Program

To change the program, press and hold the push-button until you hear the beep. The push-button is located on the bottom of the hearing aid.

- If you change to program 1, you will hear 1 beep
- If you change to program 2, you will hear 2 beeps
- If you change to program 3, you will hear 3 beeps
- If you change to program 4, you will hear 4 beeps

Whenever you turn your GOhear-on hearing aid on, it start at the last program selected and the applicable beep sequence.

You may find that certain programs are more comfortable in different situations. For instance program 1, which has less amplification, might be more comfortable in noisy environments such as restaurants. On the other hand, program 4, which has more amplification, might be more comfortable when listening to a speaker in an auditorium.

Battery Information

Battery Size:

Your GOhear-on hearing aid uses a standard size A312 hearing aid battery, which is available to purchase at your Sonetik Specialist.

Low Battery:

Your GOhear-on hearing aids are equipped with a low battery detection system, which continuously monitors battery life. When the battery becomes low, the low battery warning sequence consisting of 3 double beeps is initiated. After the warning beeps have finished, normal operation resumes. Battery replacement is recommended when you hear this low battery warning signal.

The low battery warning sequence will repeat every 10 minutes until the battery becomes critically low. The end of battery life warning sequence consisting of 6 double beeps is then triggered. The device is then muted and put into low power sleep mode. Battery replacement will then be required.


The hearing aid battery packaging usually shows a voltage. This is the idle voltage of the battery (not in use). If you measure the voltage with a voltmeter, you will see the battery voltage in use, which is much lower than when it is idle. A lower result from the voltmeter does not limit the performance or service life of the battery in any way.

Inserting the Battery

Step 1: Apply light pressure on the latch on the battery door, and swing the door open. Do not force it. Remove the battery.

Step 2: To ensure proper battery usage, please remove the tag from the battery and allow the battery to sit for two minutes before inserting the battery into the hearing aid.

Insert the battery carefully with the "+" side up, matching it to the "+" on the battery door.

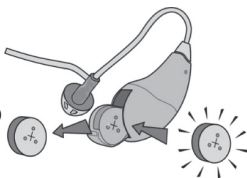Line up the + sign on the battery door to the + sign on the battery

Step 3: Swing the door into the closed position. The door should close easily. Do not force it. If there is resistance, check that the battery is inserted correctly.

The life of your hearing aid battery will vary depending on length of daily use, listening environments, and how your hearing aid is programmed. When the hearing aid battery goes dead, remove it immediately and replace it with a new battery. Batteries may be purchased from your Sonetik Specialist.

Cleaning Your Hearing Aid

The following instructions will assist in extending the life of your hearing aid.

To Clean the Exterior of the Hearing Aid:

You should inspect your hearing aid daily and clean it periodically to control build-up of dust, dirt and wax. To clean, wipe the surface very gently with a soft, dry cloth or use moist cleaning wipes made especially for hearing aids.

To Clean the Dome:

The receiver unit and dome should be cleaned regularly according to the instructions on the following pages.


Inspect the dome of the receiver unit portion of the hearing aid for any wax build-up. To clean, wipe the dome very gently with a soft, dry cloth or use moist cleaning wipes made especially for hearing aids and their accessories. The dome should be cleaned regularly and exchanged at the latest after three months.

Be very careful not to push wax into the earwax filter.

It is important that the dome fits properly into your ear. Your hearing aid is constructed from the highest quality materials, and the likelihood of an allergic reaction is low. If the dome irritates your ear in any way, contact your Sonetik Specialist.


To Clean the Microphone:

Use a brush to remove any debris from the microphone. Be careful that you don't push the debris inside the microphone. Hold the hearing aid facing downwards ensuring loose wax falls out and away and not into the microphone. If you can't remove the debris from the microphone, contact your Sonetik Specialist.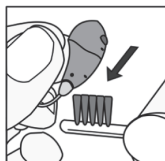


To Clean the Earwax Filter:



Use a brush to remove any earwax from the earwax filter. Hold the hearing aid sound outlet facing downwards ensuring loose wax falls out and away and not into the filter. If you can't remove the earwax, exchange the filter. It is important to replace the filter as soon as it has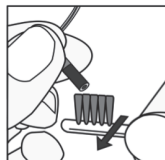

become clogged, or whenever you replace the battery. You can purchase additional filters from your Sonetik Specialist. There is a tool included in the exchange filter pack. Use this tool to remove the clogged filter from the hearing aid. Press the replacement filter onto the aperture at the end of the receiver unit. If you have difficulties replacing the earwax filter, contact your Sonetik Specialist.

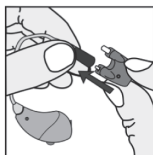The wearing of a hearing aid may accelerate the accumulation of earwax. This can easily be removed by your physician. Regular annual visits are recommended. You should not use cotton swabs to clean your ears, due to the risk of pushing the wax deeper into the ear canal.

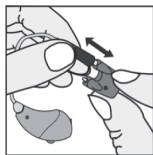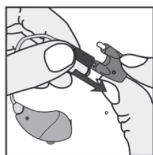Caution: Make sure that the new earwax filter is firmly in place in the receiver unit. Improper insertion can result in the earwax filter falling off and becoming lodged in the ear canal. If this happens, contact your physician immediately to have it removed. Do not attempt to remove it by yourself.

Caring For Your Hearing Aid

Hearing aids are exposed to moisture in the form of humidity and perspiration while being worn. The daily use of a dehumidifying system or drying kit approved by your hearing healthcare provider is recommended. These accessories may be purchased from your Sonetik Specialist.

Extended Storage

If it is necessary to store your hearing aids for an extended amount of time, follow the steps below for proper storage. Remove the batteries. Clean the hearing aid according to instructions in this manual. Place the hearing aids in a cool, dry place. The original container or a hearing aid dehumidifier is recommended.

Telephone Use

Your GOhear-on hearing aid allows you to use the phone as you would normally do – by holding the phone up to your ear. If you experience discomfort from the ear dome pressing in your ear canal, you may need a different size receiver unit. If this occurs, contact your Sonetik Specialist.

Troubleshooting Guidelines

No Sound/Not Loud Enough:

- Ensure the battery is inserted and the battery compartment door is fully closed.
- Replace the battery.
- Ensure that there is no wax or debris built up; clean or replace the earwax filter as instructed in this manual.
- Excessive earwax can prevent sound from entering the ear.
- Ensure the hearing aid is placed in the proper position and the dome is placed properly into the ear canal.
- Changes in your hearing loss can affect how you hear with a hearing aid.

Whistling/Feedback:

- Ensure the hearing aid is properly positioned behind the ear and the ear dome is inserted into the ear canal.
- Excessive earwax, a receiver unit that is not the proper size can cause whistling. Contact your Sonetik Specialist.

Sound is Distorted or Unclear/Intermittent Sound:

- Replace the battery.
- Your hearing aid may have poor battery contact. Return the hearing aid to your Sonetik Specialist.
- Ensure that there is no wax or debris built up; clean or replace the earwax filter as instructed in this manual.

Warranty Information

Please see your Sonetik Specialist regarding your specific warranty information. Our guarantee covers any defect in material or faulty production, except receiver units, domes and batteries. We will, at our discretion, repair or replace the hearing aid or any defective part, at our expense, within the limits of the warranty.

This warranty will become invalid if the hearing aid has been tampered with or altered in any way, or if the serial number has been altered, effaced, removed, or if any repairs have been made without authorization.

Any implied warranty shall be limited to 24 months from date of purchase. The only entity or individual who is authorized to perform the warranty obligations is the manufacturer. All in-house warranty work performed by the manufacturer will be done within a reasonable period of time. If warranty work is performed and there appears to be a defect, malfunction, or failure to conform to the warranty, the obligations will be met if brought to our attention within one month of the date of the original warranty work.

This warranty is limited to the scope set forth herein, and may only be extended with the explicit written agreement of Sonetik AG or its affiliated companies. The receipt will be required for any warranty.

This warranty does not include a promise of specific hearing improvement as this depends upon your audiological needs, upon proper fitting and adjustments by your Sonetik Specialist and upon proper use by the owner.

Returning for Service

You should visit your Sonetik Specialist if your hearing aid is in need of service.

Before planning to return your hearing aid for repair, please refer to the troubleshooting solutions on the preceding pages. In many cases, these solutions can prevent the time and cost associated with a repair.

Declaration of Conformity

Hereby Sonetik AG declares that this GOhear-on product meets the requirements of the Medical Devices Directive 93/42/EEC. The full text of the Declaration of Conformity can be obtained from Sonetik AG.

Interpretation of Serial Number

Each hearing aid has its own unique number located on the exterior of the aid. The first two digits of the serial number represent the year in which the hearing aid was manufactured.

Key to CE symbols

On the box, label and instructions for use you may encounter the following symbols, shown here with their meaning:

	Medical device directive CE mark and number of the notified body
	Manufactured by
	Date of Manufacture
	Please read user guide
	Serial number
	This hearing aid may not be thrown away as normal household waste. Please dispose of old or unused hearing aids at waste disposal sites intended for electronic waste, or give your hearing aid to your Sonetik Specialist for disposal. Proper disposal protects the environment and health.
	The products described in this user guide adhere to the requirements for an applied part of Type BF of EN 60601-1. The surface of the hearing aid is specified as application part of Type BF.

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