User Guide

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Mode d'emploi

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Sonetik
GOhear

Welcome

Welcome to a world of better hearing!

We understand that hearing loss affects one's quality of life. That is why we are dedicated to building the highest quality hearing aids. Your new hearing aid will soon become indispensable and will make your life more pleasant and more enjoyable.

Your new GOhear hearing aid has been programmed to your individual hearing loss profile to help enhance your hearing and so improve your ability to understand and follow conversations.

A period of adjustment may be necessary to become used to your new hearing aid and the restored world of sounds you will experience. The adjustment time varies for each individual and may last as little as two weeks or as long as several months. Allow yourself time to become familiar with sounds you have not heard for a long time.

Before operating your hearing aid, please read this user guide thoroughly.

Owner Information

Complete the following information for future reference:

Left:

Model _____

Base Program _____

Tube/Ear Dome _____ / _____

Serial Number _____

Right:

Model _____

Base Program _____

Tube/Ear Dome _____ / _____

Serial Number _____

Purchase Date _____

Battery Size _____

Original Warranty Expires _____

GOhear Hearing Care Professional

Name: _____

Business Name: _____

Address: _____

Postcode: _____ City: _____

Phone: _____

Table of Contents

Safety Information	4
GOhear Diagram	6
Proper Placement on the Ear	7
Left and Right Hearing Aids	8
Turning the Hearing Aid On	8
Turning the Hearing Aid Off	8
Selecting Your Program	9
Battery Information	10
Inserting the Battery	11
Cleaning Your Hearing Aid	12
Caring for Your Hearing Aid	14
Extended Storage	15
Telephone Use	15
Ear Wax	15
Troubleshooting Guidelines	16
Warranty Information	18
Returning for Service	19
Declaration of Conformity	19
Interpretation of Serial Number	19
Key to CE symbols	20
Important VAT note	20

Safety Information

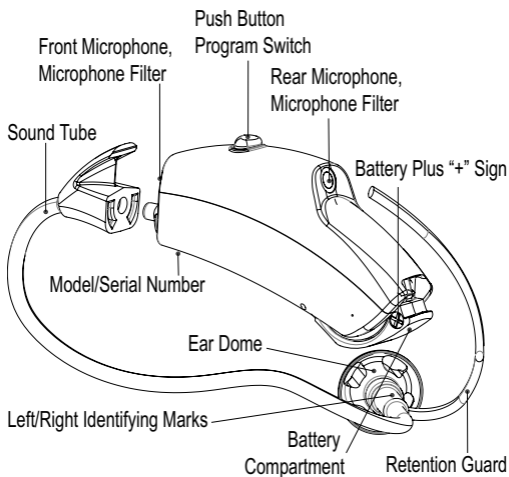
Keep the hearing aid out of the reach of children. The hearing aid contains small pieces that may be swallowed and may present a choking hazard. Contact your physician immediately if any piece is swallowed.

- Read all of the information in the User Guide before using the hearing aid.
- Always follow the directions of your GOhear Hearing Care Professional in fitting and placing the device on your ear.
- If you experience any discomfort, discontinue use, and contact your GOhear Hearing Care Professional or physician.
- Failure to change the dome according to the instructions provided could result in injury to your ear.
- Be careful when removing ear wax. Using cotton swabs or other small objects can damage the ear canal. Contact your physician.
- DO NOT push the dome of your hearing aid too deep into your ear. Place the ear dome lightly inside the outer rim of your ear canal as instructed by your GOhear Hearing Care Professional. In the rare cases in which an ear dome or other part becomes lodged in your ear canal, seek medical attention for removal.
- DO NOT leave your hearing aid on a radiator, near a stove, in a sunny window, in a car or in any other hot place. Excessive heat and cold can damage the hearing aid.
- DO NOT wear the hearing aid when using a hair dryer at home or at the beauty salon, spa or hairdresser's as this will damage the hearing aid.
- DO NOT wear the hearing aid during medical tests such as MRI, X-rays or CT scans. Different types of radiation may damage the hearing aid. Consult your medical professional prior to any medical test.

- DO NOT apply hair spray when wearing your hearing aid. This will damage the hearing aid.
- DO NOT wear the hearing aid when in contact with water, for example, when taking a bath or shower, washing your hair or swimming or during a spa treatment. Allowing water into any opening in the hearing aid will damage the hearing aid.
- DO NOT store the hearing aid in the bathroom or near a steam room. Moisture created by the steam can damage the hearing aid.
- DO NOT immerse your hearing aid in water or other liquids or put it in contact with detergents, solvents or other liquids. This will damage the hearing aid.
- DO NOT allow water or any liquid to enter any openings on the hearing aid. This will damage the hearing aid.
- DO NOT drop your hearing aid and avoid any sudden impact. This will damage the hearing aid.
- DO NOT twist or turn the sound tube when connecting it to the body of the hearing aid, as this may damage the hearing aid.
- DO NOT use the cleaning rod when the sound tube is attached to the hearing aid. This will damage the hearing aid.
- Dispose of the batteries, device, and components in accordance with local regulations. Unlawful disposal may cause environmental pollution.
- DO NOT use your hearing aid if damaged.
- DO NOT attempt to modify the shape or length of the sound tube by yourself.
- It is recommended you remove the battery and store your hearing aid overnight in a hearing aid dehumidifier.
- Always carry spare batteries for immediate replacement.
- Replace the battery as soon as you hear the low battery warning signal of 3 double beeps.

- Always care for your hearing aid as outlined in this user guide.
- Open the battery door fully when the hearing aid is not in use. This helps prevent excessive battery drain and allows proper ventilation of the hearing aid.

GOhear Diagram

Standard on GOhear1 / GOhear2 / GOhear3 / GOhear4

Proper Placement on the Ear

Your GOhear hearing aid should fit comfortably and easily over the top of your ear. Your GOhear Hearing Care Professional will provide instruction on proper insertion and removal.

Step 1: Place the hearing aid over the top of your ear.

Step 2: With your index finger and thumb, hold the sound tube where it attaches to the ear dome and gently push the dome into your ear until it is just inside your ear canal.

Step 3: Place the retention guard of the sound tube into the bow of your concha.

Left and Right Hearing Aids

If you are wearing hearing aids on both ears, it is important to use the correct device in each ear. The sound tubes are marked with «red» lettering for the right ear and «blue» lettering for the left ear.

Turning the Hearing Aid On

To turn your hearing aid «on», you simply need to close the battery door after inserting a battery.

Turning the Hearing Aid Off

Apply light pressure on the notch on the battery door, and open until the battery is visible. Your hearing aid is now «off».

When the hearing aid is not in use, it is recommended that you always open the battery door fully. This helps prevent excessive battery drain and allows proper ventilation of the hearing aid.

Selecting Your Program

Refer to the owner information page to find the base program for your right and left GOhear hearing aids.

Your GOhear hearing aid resets to program 1 whenever the hearing aid is turned off and back on.

To change the program, press and hold the push-button until you hear the beep. The push-button is located on the top of the hearing aid.

- Press the push-button once to select program 2 (you will hear 2 beeps).
- Press the push-button again to select program 3 (you will hear 3 beeps).
- Press the push-button again to select program 4 (you will hear 4 beeps).
- Press the push-button again to go back to program 1 (you will hear 1 beep).

You may find that certain programs are more comfortable in different situations. For instance program 1, which has less amplification, might be more comfortable in noisy environments such as restaurants. On the other hand, program 4, which has more amplification, might be more comfortable when listening to a speaker in an auditorium.

Battery Information

Battery Size

Your GOhear hearing aid uses a standard size A312 hearing aid battery, which is available to purchase at your GOhear Hearing Care Professional.

Low Battery

Your GOhear hearing aids are equipped with a low battery detection system, which continuously monitors battery life. When the battery becomes low, the low battery warning sequence consisting of 3 double beeps is initiated. After the warning beeps have finished, normal operation resumes. Battery replacement is recommended when you hear this low battery warning signal.

The low battery warning sequence will repeat every 10 minutes until the battery becomes critically low. The end of battery life warning sequence consisting of 6 double beeps is then triggered. The device is then muted and put into low power sleep mode. Battery replacement will then be required.

The hearing aid battery packaging usually shows a voltage. This is the idle voltage of the battery (not in use). If you measure the voltage with a voltmeter, you will see the battery voltage in use, which is much lower than when it is idle. A lower result from the voltmeter does not limit the performance or service life of the battery in any way.

Inserting the Battery

Step 1: Apply light pressure on the notch on the battery door, and swing the door open. Do not force it. Remove the battery.

Step 2: To ensure proper battery usage, please remove the tag from the battery and allow the battery to sit for one-two minutes before inserting the battery into the hearing aid. Insert the battery carefully with the «+» side up, matching it to the «+» on the battery door.

Step 3: Swing the door into the closed position. The door should close easily. Do not force it. If there is resistance, check that the battery is inserted correctly.

The life of your hearing aid battery will vary depending on length of daily use, listening environments, and how your hearing aid is programmed. When the hearing aid battery goes dead, remove it immediately and replace it with a new battery. Batteries may be purchased from your GOhear Hearing Care Professional.

Cleaning Your Hearing Aid

The following instructions will assist in extending the life of your hearing aid.

To Clean the Exterior of the Hearing Aid:

You should inspect your hearing aid daily and clean it periodically to control build-up of dust, dirt and wax. To clean, wipe the surface very gently with a soft, dry cloth or use only moist cleaning wipes made especially for hearing aids.

To Clean the Sound Tube and Ear Dome:

The sound tube delivers the amplified sound from the hearing aid into the ear. It is important that the sound tube and the ear dome fit properly in your ear. The materials used in your hearing aid are of the highest quality available, and the likelihood of an allergic reaction is low. If the sound tube or the ear dome irritates your ear in any way, contact your GOhear Hearing Care Professional.

The tube and dome should be cleaned regularly according to the instructions on the following pages.

Use a damp cloth to clean the exterior of the sound tube and dome. Do not rinse or submerge these parts in water, as there is a risk of damage if a water droplet becomes trapped in the sound tube. If this should occur, it will prevent sound from coming through the tube and may be harmful to the hearing aid's electronics.

Remove the sound tube from the hearing aid by gently twisting the tube 90° and pulling it straight off and away from the hearing aid.

Use the cleaning rod (provided in the storage case) to clean the sound tube and dome. Insert the cleaning rod in the opening where the tube attaches to the body of the hearing aid, lightly «push» the cleaning rod through the sound tube, and pull it out through the dome. Wipe off the cleaning rod and store it for future use.

Re-attach the sound tube onto the body of the hearing aid by holding the sound tube in one hand and the hearing aid in the other hand. Line up the sound tube and the

connecting end of the hearing aid; push the sound tube straight onto the hearing aid body. There is no need to rotate the sound tube when reattaching.

Caring for Your Hearing Aid

The sound tube and the dome may need to be changed if the sound tube becomes stiff, brittle, kinked or damaged. Only your GOhear Hearing Care Professional should change the ear dome. This is to prevent the dome from detaching from the sound tube during insertion into the ear. If your GOhear Hearing Care Professional instructs you to change the domes yourself, make sure they are securely fastened to the sound tube before inserting them in your ear.

Hearing aids are exposed to moisture in the form of humidity and perspiration while being worn. The daily use of a dehumidifying kit with a drying capsule is recommended. Your GOhear Hearing Care Professional may carry these accessories for you to purchase or recommend a place of purchase to you.

Extended Storage

If it is necessary to store your hearing aid for an extended amount of time, follow the steps below for proper storage.

1. Remove the battery.
2. Clean the hearing aid according to the instructions on page 12.
3. Place the hearing aid in a cool, dry place. It is recommended you use either the original container or a hearing aid dehumidifier.

Telephone Use

Your GOhear hearing aid allows you to use the phone as you would normally do – by holding the phone up to your ear. If you experience discomfort from the ear dome pressing in your ear canal, you may need a different size sound tube. If this occurs, contact your GOhear Hearing Care Professional.

Ear Wax

The wearing of a hearing aid may accelerate the accumulation of ear wax. This can be easily removed by your physician. You should not use cotton swabs to clean your ears, as you may risk pushing the wax deeper into the ear canal.

Troubleshooting Guidelines

No Sound

- Ensure the battery is inserted and the battery compartment door is fully closed.
- Replace the battery.
- Ensure there is no wax or debris in the sound tube or ear dome. If needed, clean according to the instructions on page 12.

Not Loud Enough

- Replace the battery.
- Ensure there is no wax or debris in the sound tube or ear dome. If needed, clean according to the instructions on page 12.
- Ensure the hearing aid is properly positioned behind the ear and the ear dome is inserted into the ear canal.
- Excessive earwax can prevent sound from entering the ear. Contact your physician.
- Changes in your hearing loss can affect how you hear with a hearing aid. Contact your GOhear Hearing Care Professional if you suspect any change in your hearing status.

Whistling/Feedback

- Ensure the hearing aid is properly positioned behind the ear and the ear dome is inserted into the ear canal.
- Excessive earwax and sound tubes that are not the proper size can cause whistling. Contact your GOhear Hearing Care Professional.

Sound is Distorted or Unclear

- Replace the battery.
- Your hearing aid may have poor battery contact. Return the hearing aid to your GOhear Hearing Care Professional.

Intermittent Sound

- Replace the battery.
- Ensure there is no wax or debris in the sound tube or ear dome. If needed, clean according to the instructions on page 12.
- Your hearing aid may have poor battery contact. Return the hearing aid to your GOhear Hearing Care Professional.

Static Noise

- Replace the battery.
- If you are near an electromagnetic field (i.e. near a computer screen or fluorescent lights), step away to see if the noise clears.
- Your hearing aid may have poor battery contact. Return the hearing aid to your GOhear Hearing Care Professional.

Warranty Information

Please see your GOhear Hearing Care Professional regarding your specific warranty information. Our guarantee covers any defect in material or faulty production, except tubes, domes and batteries. We will, at our discretion, repair or replace the hearing aid or any defective part, at our expense, within the limits of the warranty.

This warranty will become invalid if the hearing aid has been tampered with or altered in any way, or if the serial number has been altered, effaced, removed, or if any repairs have been made without authorization.

Any implied warranty shall be limited to 24 months from date of purchase. The only entity or individual who is authorized to perform the warranty obligations is the manufacturer. All in-house warranty work performed by the manufacturer will be done within a reasonable period of time. If warranty work is performed and there appears to be a defect, malfunction, or failure to conform to the warranty, the obligations will be met if brought to our attention within one month of the date of the original warranty work.

This warranty is limited to the scope set forth herein, and may only be extended with the explicit written agreement of Sonetik AG or its affiliated companies. The receipt will be required for any warranty.

This warranty does not include a promise of specific hearing improvement as this depends upon your audiological needs, upon proper fitting and adjustments by your GOhear Hearing Care Professional and upon proper use by the owner.

Returning for Service

You should visit your GOhear Hearing Care Professional if your hearing aid is in need of service.

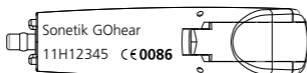
Before planning to return your hearing aid for repair, please refer to the troubleshooting solutions on the preceding pages. In many cases, these solutions can prevent the time and cost associated with a repair.

Declaration of Conformity

Hereby Sonetik AG declares that this GOhear product meets the requirements of the Medical Devices Directive 93/42/EEC. The full text of the Declaration of Conformity can be obtained from Sonetik AG.

Interpretation of Serial Number

Each hearing aid has its own unique number located on the exterior of the aid. The first two digits of the serial number represent the year in which the hearing aid was manufactured.

Key to CE symbols

On the box, label and instructions for use you may encounter the following symbols, shown here with their meaning:

	Medical device directive CE mark and number of the notified body
	Manufactured by
	Date of Manufacture
	Please read user guide
	Serial number
	This hearing aid may not be thrown away as normal household waste. Please dispose of old or unused hearing aids at waste disposal sites intended for electronic waste, or give your hearing aid to your Hearing Care Professional for disposal. Proper disposal protects the environment and health.
	The products described in this user guide adhere to the requirements for an applied part of Type BF of EN 60601-1. The surface of the hearing aid is specified as application part of Type BF.

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